



TO INSTALL

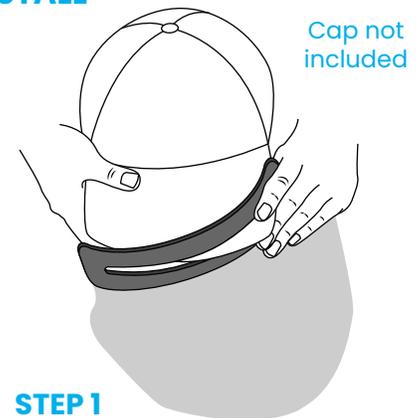


STEP 1

Slide one corner of brim into brim mount slot at an angle

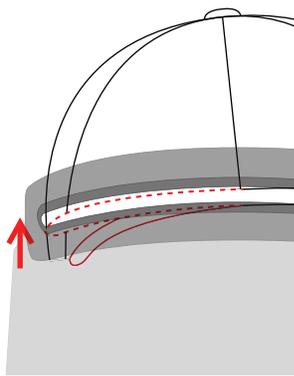


STEP 2

Flex opposite side of brim up and slide into brim mount slot

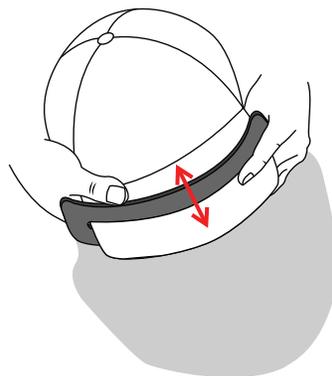

STEP 3

Slide brim mount along brim adjusting shield to a comfortable distance away from face. This can help further reduce the possibility of fogging.

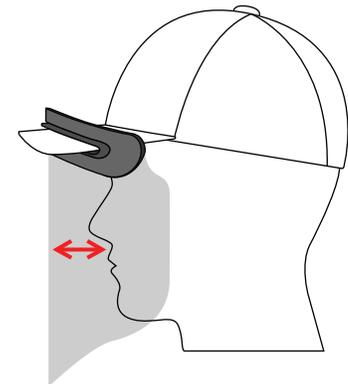

CLEANING & CARE FOR YOUR FACESHIELD

- Disinfect the shield using common household disinfectants or clean using liquid antibacterial soap, warm water and a soft cloth.
- Only use cleaners and disinfectants that are compatible with the CrewSafe Face Shield. For a detailed list visit our FAQs at tuuw.com.
- Thoroughly dry after cleaning to avoid water spots.
- Do not wash, wipe or dry the shield with rigid or abrasive materials such as rough cloths, scrub pads or paper towels.
- Do not use around intense heat sources or open flame.
- Replace the shield when it becomes damaged, contaminated, or clarity is degraded.



Video available at tuuw.com

For more information visit our FAQs at tuuw.com

NOTICE OF DISCLAIMER OF LIABILITY Spellbound Development Group, Inc. does not make any representations regarding the effectiveness of specific disinfectants, cleaning agents or chemicals against infectious agents or contagions. Spellbound Development Group, Inc. does not warrant, expressed or implied that our products eliminate losses due to injuries, health hazards, and accidents.